


The outlook ...

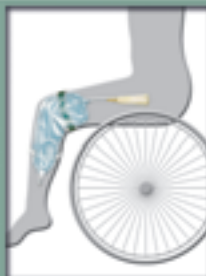

... how to control incontinence

Main Catalogue

Continence Systems

2009/2010



SAUER
CONTINENCE

Manfred Sauer Foundation

Meaning in life



In 2007 we celebrated the 25th company anniversary here at Lobbach in Baden. You can guess from the outlines of the buildings (above left) that we have remained a medium-sized, independent business that is successfully engaged in helping customers in coming to terms with incontinence problems.

Further to the well-established and proven urinary condoms we have now added a single use catheter, which is furnished with a novel soft tip that finds its way to the bladder more easily and without any risk of injuries. Like all the components of our continence system this catheter is manufactured on our premises in Germany and thus provides new jobs.

Not being spared by the ravages of time myself, I started wondering what was to become of the Manfred Sauer Company? How can the jobs in Lobbach and the pursuance of the individual approach be secured? Last but not least

how can the economic success benefit those that brought it about?

Our Foundation concept is the positive answer to these questions. As the owner of our company it safeguards the production site and attends particularly to people with paraplegia seeking (re-)orientation.

On 21. May 2006 the Manfred Sauer Foundation Centre (above right) was opened and since has received a very positive response not only from wheelchair users, but also from the local population. With their lively interest in the widespread offers of the Foundation they ensure that this institution doesn't turn into a pure disabled facility, but a meeting place for all people regardless of physical impairment.

Yours,
Manfred Sauer

Please visit our website for further information: www.manfred-sauer.com

The principle ...

- Urinary condoms
- Urine collection bags



**...of being supplied with
contemporary urine drainage system
for wheelchair-users, the able-bodied, children,
and bedridden people**



Sterile urine collection bag

On stock and available on request: sterile and individually packed ► page 33.



Exchange

The products in this catalogue can only be exchanged if originally packed. Exempt are the ones marked with the symbol ☒, products without original packaging and all custom-made products.

Contents

INFO	Urinary condoms	4
INFO	SAUER skin adhesive	5
	Self adhesive condoms	
	- SAUER-Comfort	6
	- without buffer zone	8
	2-pieces urinary condoms	
	- Synthetic condoms	10
	- Latex condoms	11
	Condom-Adhesion	
	SAUER skin adhesive	12
	Special skin adhesive	13
	Double-sided adhesive tapes	14
	Accessories for condoms	15
	Adhesive enhancers and skin care.....	16
	Supplements	17
INFO	Urine collection bags	18
	- Custom products	21
	SAUER bendi bags	
	- no adapter.....	22
	- with fixed adapter.....	23
	- with push-on adapter.....	24
	SAUER leg bags	25
	Leg bags	27
	SAUER sport bags	28
	SAUER discreet bags.....	29
	SAUER functional bags.....	30
	Night bags.....	31
	Sterile urine collection bags	33
	Bag fixation	
	SAUER cotton cover	34
	Legs straps and waist belts	35
	Leg bag holsters	36
	Accessories for night bags	37
	Accessories and spare components	38
	Special urine drainage systems	
	SAUER drip urinal	40
	Pocket size urinal Uribag	41
	Latex urinal.....	42
INFO	KIC-System for ISK	44
	KIC-System	45
	Disinfectants.....	46
	Literature	47
	Sauer Continence World-wide.....	48
	Index.....	50

The urinary condoms have been designed and manufactured in our factory at Lobbach in Germany and have the necessary characteristics to ensure a secure, reliable and simple handling and, last but not least, a high wearing comfort.

Urinary condoms

Material: Synthetic material or latex

Fixation: Self-adhesive or with skin adhesive/adhesive strips



Irrespective of their specifications, all our urinary condoms have the following properties in common:

11 sizes: You can get an exact fit as we have eleven finely graded sizes (diameter 18–40 mm). The better the fit, the safer the adhesive contact.

Synthetic material: The condoms and skin adhesive are latex-free. This prevents irritations and allergies that can occur with latex.

The high elasticity of the material: The extremely high elasticity of the material allows for large amounts of spontaneous urine and, at the same time, a residue free drainage. Therefore no pooling occurs around the tip of the penis, being more hygienic and reducing the risk of infection.

Material strength: Made from very thin but resistant materials which ensure a snug fit and a reliable adhesion under all circumstances.

Double-dipped tip: To prevent tearing.

Connecting tube: The condom and connecting tube are manufactured all in one piece or are mounted with a nipple and are suitable for all urine bags with or without adapter.

Self-adhesive film: As it is inside the condom, this allows you to put it on quicker and more easily. It also gives a reliable adhesion between the condom and the skin.

100 % individually tested
To guarantee maximum safety all urinary condoms are tested *individually* for impermeability and tear strength.

Advice, samples, custom products
Have you got questions regarding urinary condoms? Would you like us to supply you with free samples? Do you have individual requirements (extra charge)? Please turn to your local continence adviser for further information ► page 48.

Self adhesive condoms
SAUER-Comfort – Standard► page 6
SAUER-Comfort – Special productions► page 7
Self-adhesive condoms without buffer zone► page 8
2-piece urinary condoms
Synthetic condoms – Anti-Allergic► page 10
Latex condoms – Original► page 11
Latex condoms – Extra thin► page 11

Choosing the adequate urinary condom

As a first step you should use the measuring tape (► page 15) to determine the required condom size. Because of the easy handling you should begin with the self-adhesive condoms SAUER Comfort.

For further alternatives or if you have any questions regarding the use of skin adhesives please turn to your local continence adviser ► page 48.

Articles made of latex

Many of our customers confirm a long-time use of SAUER urinary condoms made of latex without any reactions to the skin. Another point in favour of latex is the fact that it has undisputed advantages in comparison to synthetic material: As a natural product obtained from the rubber tree, its availability is nearly unlimited, and additionally it is environmentally friendly - both in terms of production and disposal.

One must keep in mind that synthetic rubber and silicone contain a multitude of chemical substances whose effects are not sufficiently known yet. The production and working of synthetic rubber, a product made of petroleum, is more complex and, because of its high content of solvents, more problematic in production and for the environment.

If you have concerns about the use of latex-containing condoms or skin adhesives or if you suffer from a hypersensitivity towards latex, you can change to latex-free products made of synthetic materials.

You will find more information about this topic in our "Continence Systems Tutorial". Please ask for your personal copy. It is also available on our website:

www.sauercontinence.com

SAUER skin adhesive

Liquid skin adhesives provide the safest bonding between skin and condom. Unlike the self-adhesive condoms they allow an individual handling; this means that the user himself can determine the required quantity of adhesive, the length of the adhesive zone and the size of the buffer zone. For more than 30 years SAUER skin adhesives have been used successfully for the fixation of urinary condoms.

Eudermic composition: We offer the skin adhesive in different compositions that match individual skin properties. The SAUER Original skin adhesive is composed of natural rubber, mastic as the adhesive component, heptane as solvent to determine the viscosity, lanolin (purified wool fat) to replenish the oils in the skin and zinc oxide for skin protection.

Adhesion effect: This varies according to the composition of the different skin adhesives used. The skin adhesives are as follows:

- Skin adhesives 50.01 and 50.03 provide adhesion which should in almost all cases lasts round-the-clock.
- The reduction of skin caring agents increases the adhesive strength (50.05 and 50.00).
- By adding resin, the adhesive strength can be increased further (50.20 and 50.22).

Water resistance: The skin adhesive allows you to wear the urinary condom in the bath or when swimming without impairing the adhesion strength.

Skin friendliness: Additional substances with skin care properties allow to use the urinal "around the clock".

Removal: Removal of the condom does not leave adhesive residue on the skin since the condom and the skin adhesive complement each other.

With tube: The pipette applicator allows a simple and clean application of the adhesive.

SAUER skin adhesive	► page 12
Special skin adhesive	► page 13
Double-sided adhesive tapes	► page 14
Accessories for condoms	► page 15
Adhesive enhancers and skin care.....	► page 16

Self-adhesive condoms SAUER-Comfort

i In order to meet as many of the users' more specific requirements as possible, apart from the standard version (see below) we offer three special versions (► page 7) of the self-adhesive SAUER-Comfort condoms. The differences lie in the length of the adhesive zone and in the adhesive strength.

Condom and adhesive are made of latex free materials. Condom and connecting tube are manufactured all in one piece. To be used with all urine collection bags with and without adapter. Available in 11 sizes.

SAUER-Comfort – Standard

Total length approx. 175 mm, buffer zone 75 mm, standard adhesive.

30 pieces

Size Ø 18 mm

Order no. 97.18

20 mm

97.20

22 mm

97.22

24 mm

97.24

26 mm

97.26

28 mm

97.28

30 mm

97.30

32 mm

97.32

35 mm


97.35

37 mm

97.37

40 mm

97.40

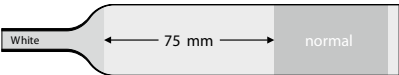

latex free



*Sauer
Comfort*

~~18~~

~~20~~

~~22~~

~~24~~

~~26~~

~~28~~


~~30~~

~~32~~

~~35~~

~~37~~


~~40~~

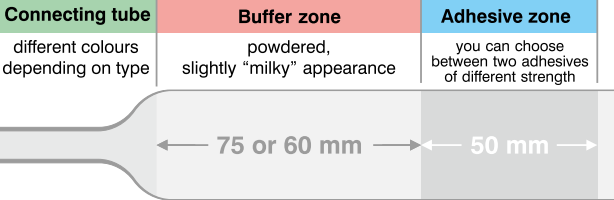

The connecting tube is directly dipped to the condom and coloured differently depending on the type of condom. The area between the connecting tube and the adhesive zone is powdered and serves as buffer zone with a length of either 75 or 60 mm. It is followed by the adhesive zone with option for two adhesives different in strength.

Tip: In order to get used to the handling, you can order a thumb-sized condom.

This is a self-adhesive condom with a diameter of 22 mm. You can use it to test the separate steps of the adhesion procedure on your thumb and find out how to deal with problems if they arise, how to avoid crinkles and learn where the adhesive zone begins.

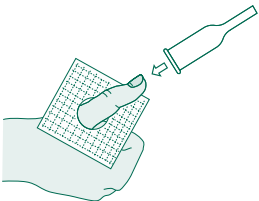


SAUER-Comfort – Blue

Total length approx. 175 mm, buffer zone 75 mm, stronger adhesive.

30 pieces

Size Ø	Order no.
18 mm	97.7501.18
20 mm	97.7501.20
22 mm	97.7501.22
24 mm	97.7501.24
26 mm	97.7501.26
28 mm	97.7501.28
30 mm	97.7501.30
32 mm	97.7501.32
35 mm	97.7501.35
37 mm	97.7501.37
40 mm	97.7501.40

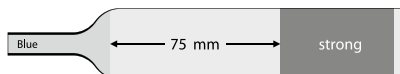


97.7501.xx

SAUER-Comfort – Green

Total length approx. 160 mm, buffer zone 60 mm, standard adhesive.

30 pieces

Size Ø	Order no.
18 mm	97.6000.18
20 mm	97.6000.20
22 mm	97.6000.22
24 mm	97.6000.24
26 mm	97.6000.26
28 mm	97.6000.28
30 mm	97.6000.30
32 mm	97.6000.32
35 mm	97.6000.35
37 mm	97.6000.37
40 mm	97.6000.40

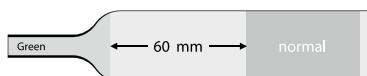

97.6000.xx




97.6001.xx

SAUER-Comfort – Red

Total length approx. 160 mm, buffer zone 60 mm, stronger adhesive.

30 pieces

Size Ø	Order no.
18 mm	97.6001.18
20 mm	97.6001.20
22 mm	97.6001.22
24 mm	97.6001.24
26 mm	97.6001.26
28 mm	97.6001.28
30 mm	97.6001.30
32 mm	97.6001.32
35 mm	97.6001.35
37 mm	97.6001.37
40 mm	97.6001.40

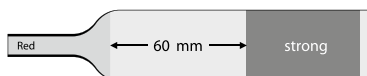**97.6000.30**

Condom size
diameter in mm

Adhesive

00 = standard adhesive strength
01 = greater adhesive strength

Length of buffer zone

75 = standard (75 mm)
60 = shortened (60 mm)

Self-adhesive condoms SAUER-Comfort

With the 3 special versions, you can identify the two distinguishing features (buffer zone and adhesive strength) by having a look at the order no.

Self-adhesive condoms without buffer zone

i Apart from the self-adhesive condoms SAUER-Comfort with the typical buffer zone, there are further models available with a more rigid funnel that can help to apply the condom e.g. when circumcised.


With the condoms *UltraFlex*, *Pop-On*, and *Wide-Band* the adhesive zone begins directly behind the funnel, where the condom is applied to the penis. The illustrations show the position and the length of the adhesive zone.

Please note that these condoms don't have an (elastic) buffer zone. The buffer zone can be necessary to cope with larger quantities of spontaneous urine as it can store the urine until it drains into the urine bag. However, some users regard the stronger material and the more rigid funnel as advantageous.

Self-adhesive condoms – UltraFlex **NEW**

Silicone condom with synthetic adhesive. Total length approx. 145 mm, the adhesive zone begins directly behind the funnel and has a length of approx. 45 mm. To be used with all urine collection bags furnished with an universal adapter. Individually packed.

30 pieces

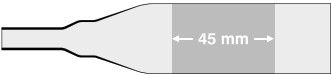

Size Ø 25 mm

29 mm

32 mm

36 mm

41 mm

Small

Medium

Intermediate

Large

X-Large

Order no. **93.25**

93.29

93.32

93.36

93.41



latex free




93.25
:
93.41

Self-adhesive condoms – Wide-Band

Silicone condom with synthetic adhesive. Total length approx. 145 mm, the adhesive zone begins directly behind the funnel and has a length of approx. 70 mm. To be used with all urine collection bags furnished with an universal adapter. Individually packed.

30 pieces

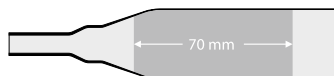

Size Ø 25 mm

29 mm

32 mm

36 mm

41 mm

Small

Medium

Intermediate

Large

X-Large

Order no. 94.25

94.29

94.32

94.36

94.41



latex free

94.25


⋮

94.41

Self-adhesive condoms – Pop-On

Silicone condom with synthetic adhesive. Total length approx. 100 mm, the adhesive zone begins directly behind the funnel and has a length of approx. 40 mm. To be used with all urine collection bags furnished with an universal adapter. Individually packed.

30 pieces

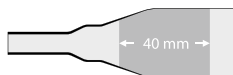

Size Ø 25 mm

29 mm

32 mm

36 mm

41 mm

Small

Medium

Intermediate

Large

X-Large

Order no. 95.25

95.29

95.32

95.36

95.41



latex free

95.25

⋮

95.41

2-piece condoms

Synthetic condoms – Anti-Allergic

Made of synthetic material. Nearly all varieties of the skin adhesives and the double-sided adhesive tapes are suitable for their fixation. Total length approx. 200 mm, condom and connecting tube are manufactured all in one piece. To be used with all urine collection bags furnished with and without adapter. Available in 11 sizes.

30 pieces

Size Ø	Order no.
18 mm	56.18
20 mm	56.20
22 mm	56.22
24 mm	56.24
26 mm	56.26
28 mm	56.28
30 mm	56.30
32 mm	56.32
35 mm	56.35
37 mm	56.37
40 mm	56.40

Anti *latex free*
Allergic

latex free



~~18~~

~~20~~

~~22~~

~~24~~

~~26~~

~~28~~

~~30~~

~~32~~

~~35~~

~~37~~

~~40~~



Adhesion alternatives

To get the best adhesion between Anti-Allergic condoms and skin adhesive, please turn to your local continence adviser for further information ► page 48.



Latex condoms – Original

Made from natural latex, practically free from chemicals. Suitable for nearly all skin types. Total length approx. 215 mm, condom and connecting tube are mounted with a nipple. To be used with all urine collection bags furnished with and without adapter. Available in 11 sizes.

30 pieces

Size Ø 18 mm

Order no. 53.18

20 mm

53.20

22 mm

53.22

24 mm

53.24

26 mm

53.26

28 mm

53.28

30 mm

53.30

32 mm

53.32

35 mm

53.35

37 mm

53.37

40 mm

53.40

*Original
well established!*

Latex condoms – Extra thin

Alternative made with an extra thin latex film. The condoms fit like a “second skin”. Total length approx. 250 mm, condom and connecting tube are mounted with a nipple. To be used with all urine collection bags furnished with and without adapter. Available in 3 sizes.

30 pieces

Size Ø 26 mm

Order no. 53.26.D

30 mm

53.30.D

35 mm

53.35.D



~~18~~

~~20~~

~~22~~

~~24~~

~~26~~

~~28~~

~~30~~

~~32~~

~~35~~

~~37~~

~~40~~



SAUER skin adhesive

SAUER skin adhesive – Original

The skin adhesive which can be used without problem in almost all cases. It provides an absolutely reliable adhesion.

2 tubes, each 28 g

Order no. 50.01

SAUER skin adhesive – Without lanolin

Occasionally, lanolin may cause allergic skin reactions. This adhesive has the same properties as the “Original” – but is free of lanolin. However, it does not supplement the skin’s natural oils!

2 tubes, each 28 g

Order no. 50.03

SAUER skin adhesive – 50 %

Based on the “Original”, but with a 50 % reduction in skin care components (zinc oxide and lanolin). This increases the adhesive qualities.

2 tubes, each 28 g

Order no. 50.05

SAUER skin adhesive – Pure

Based on the “Original”, but without the skin care components (zinc oxide and lanolin). Strong adhesive qualities – you should therefore pay attention to the condition of your skin.

2 tubes, each 28 g

Order no. 50.00

SAUER skin adhesive – 2 % resin

Based on the “Original”, but with 2 % resin to increase the adhesive quality. Therefore, pay particular attention to the condition of your skin.

2 tubes, each 28 g

Order no. 50.20

SAUER skin adhesive – 12 % resin

Based on the “Original”, but with 12 % resin to give high adhesive quality. Therefore, pay particular attention to the condition of your skin.

2 tubes, each 28 g

Order no. 50.22



Choosing the right skin adhesive

It is simply impossible to recommend one type of adhesive for all situations, because every skin type reacts differently. You should first try our well-established “Original”. Only if you are not getting satisfactory results in spite of correct application you should try an alternative. If you apply an adhesive with stronger adhering properties you should carefully check the skin condition.

Please turn to your local continence adviser for further information ► page 48.



Special skin adhesive



The following skin adhesives may provide a solution for special skin conditions or when you have serious adhesion problems. Please contact your continence adviser ► page 48 about the suitability of the special skin adhesives.

White skin adhesive

Based on the “Original” 50.01, but with a modified ratio of the latex, hexan and zinc oxyde components. In tube with applicator.

2 tubes each 28 g

Order no. 50.31

SAUER skin adhesive – Synthetic *latex free*

Synthetic adhesive containing no latex or silicone. Comes in a glass bottle with a brush applicator in the lid or in a tube with a pipette applicator.

Adhesive in glass bottle, 45 ml

Order no. 50.36

2 tubes each 28 g

Order no. 50.36-2

U-Bond skin adhesive *latex free*

Liquid silicone adhesive, in glass bottle. If the adhesive thickens due to evaporation of the solvent, you can increase the viscosity by adding the solvent. Both Adhesive and thinner are latex free.

Adhesive in glass bottle, 44 ml

Order no. 50.37

Thinner in glass bottle, with applicator, 50 ml

Order no. 50.39

Medical Adhesive B spray *latex free*

Liquid silicone adhesive, aerosol.

150 ml

Order no. 50.40

Adhesive remover pads

To remove adhesive residue. Especially for silicone-based adhesives.

50 pads

Order no. 50.38



50.31



50.36



50.36-2



50.37



50.39



50.40



50.38

Double-sided adhesive tapes

Double-sided adhesive tape has been found to be helpful in the care of the elderly, for bedridden patients and in intensive care units. It is also good for children and for all other cases where there is inadequate skin area for adhesion.

The adhesion strength of the various types of double-sided tape depends very much on the skin reaction of each individual. Therefore, we cannot know which tape is best for you. We would be more than happy to send you samples, so you can test which one suits you best.

Double-sided adhesive tape – Extra thin

Hydrocolloid tape, particularly skin friendly, very elasticated.

30 tapes	120 × 15 mm	Order no. 50.14-M
	150 × 15 mm	Order no. 50.14

Double-sided adhesive tape – Foam strip

PVC-foam-material. This tape increases the circumference of the penis through the thickness of the material, so you may need a larger condom size.

30 tapes	150 × 20 mm	Order no. 50.04
----------	-------------	------------------------

Double-sided adhesive tape – Urihesive

Hydrocolloid tape, particularly skin friendly, elasticated.

15 tapes	120 × 15 mm	Order no. 50.06
----------	-------------	------------------------



50.14
Handy box for easy removal



Alternatives

If by any chance none of the above skin adhesives or adhesive tapes suit your requirements, please turn to your local continence adviser for further information ► page 48.

Accessories for condoms

Measuring tape

To determine the exact size. Free of charge.

1 piece

Order no. 99.897



99.897

SAUER Protective cloths

Prevents the pubic hair from sticking to the adhesive when applying a condom. To be sure of a close fit, the hole in the protective cloth is 1mm smaller than the corresponding condom size. Made of natural cotton.

30 pieces

for condom size Ø 18 mm

Order no. 52.70-18

20 mm

52.70-20

22 mm

52.70-22

24 mm

52.70-24

26 mm

52.70-26

28 mm

52.70-28

30 mm

52.70-30

32 mm

52.70-32

35 mm

52.70-35

37 mm

52.70-37

40 mm

52.70-40



52.70-18

:

52.70-40

Scissors

Safety scissors for removing the collar thus preventing discomfort or pressure sores.

1 piece

Order no. 53.50



53.50

Preventox skin protection

i Because of its high content of alcohol, Preventox has cleaning, degreasing and disinfecting properties and forms an elastic film on the skin, which allows the skin to breathe. Particularly the cleansing of the skin and the formation of the film usually improve the adhesive strength when you apply a urinary condom (self-adhesive, with skin adhesive of adhesive strip). In addition to this, it protects the skin.

Preventox single wipes

Easy handling and especially hygienic.

50 wipes

Order no. 50.50

Preventox fluid

With applicator.

50 ml

Order no. 50.52

Spray head

To go with bottle 50.52.

1 piece

Order no. 50.53

Preventox fluid roller

Use like an under arm roll-on deodorant. Particularly suitable for users with limited dexterity.

50 ml

Order no. 50.58

SkinPrevent single wipes

An alternative to Preventox.

50 wipes

Order no. 50.51



50.50



50.52



50.53



50.58



50.56



50.51

Washing gel Uroclean

Transparent gel for the daily care of the genital area, particularly in cases of incontinence, catheterisation and use of urinary condoms. Uroclean cleans and cares for the skin; it cools and refreshes, deodorizes and has antibacterial properties. It is friendly to the skin, alkali-free (pH-value 5) and biologically decomposable.

Tube, 100 ml

Order no. 50.56

Dispenser, 250 ml

Order no. 50.57

Refill, 1 litre (not illustrated)

Order no. 50.59



50.57



Cranberry concentrate

NEW


The cranberry (*Vaccinium macrocarpon*), which is related to the native mountain cranberry, is a natural aid to strengthen the whole organism and keep it healthy, especially the urinary passage. This is due to a high percentage of natural antioxidants and secondary plant compounds.

Cranberry concentrate is a nutritional supplement – it is not a medication for the treatment of illnesses or symptoms and no replacement for a balanced and varied diet.

Due to long years of experience and scientific studies, the effects of cranberries are now well known.

Anti-adhesive effect: the proanthocyanidins contained in cranberries, prevent bacteria from settling due to special adhesive features in the bladder wall. This includes *Escherichia coli*, the most frequent trigger for urine infections. The bacteria are washed out with the urine, therefore preventing an infection.

Acidification of the urine: Cranberries have a low pH value and therefore taste sour. As their contents are distributed throughout the kidneys, the consumption of cranberries and/or cranberry juice acidifies the urine. An acidic environment prevents the multiplication of bacteria, which prevents urinary passage infections.

Antioxidant effect: The anthocyanins (polyphenol, flavonoids) and vitamin C in the cranberries have the ability to protect the cells in the human body from free radicals (antioxidant effect) and therefore strengthen the body's own defences and fight against cell damage. Furthermore, vitamin C strengthens the general defence situation.

Cranberry juice concentrate

Nutritional supplement to support a healthy bladder function. It is extracted by a gentle process using fresh North American cranberries. It contains no wheat, sugar, artificial colours or preservatives. Every capsule contains 400 mg cranberry concentrate and 60 mg vitamin C.

The effect of a capsule lasts for approx. 10 hours. For a “round the clock protection”, it is recommended to take a capsule every morning and evening.

Container with 100 capsules

Order no. 110.80



Further information about Cranberries: www.cranberryinstitute.org

The urine collection bags have been developed and are produced by us. They are manufactured on high-frequency welding machines at our premises at Lobbach. Their different shapes, sizes and various features make it possible to meet the particular needs and requirements of most users.

Urine collection bags

Bendi bags, leg bags, sport bags, discreet bags, functional bags, night bags



SAUER bendi bags

The bendi bag has been designed especially for wheelchair users. They are adapted to the sitting position and offer a high degree of wearing comfort, flexibility and security. Due to their great variety of features, they fit every user according to his individual need.

High capacity: Available in 1.3 litre, these give you more independence on long journeys and at work without the worry of having to find accessible toilets. The 0.7 litre bags are for wheelchair users that do not require the extra capacity or may be of a smaller build persons. Many able bodied and semi-ambulant people also use the 0.7 litre bags.

Leg hugging shape: Fitted with welded angled inlet tube, this is essential for a bendi bag adapted to the sitting position. It can be worn comfortably on the leg, is inconspicuous, has a reliable urine outlet and is easy to empty.

SAUER leg bags

Their design makes them particularly suitable for able-bodied people, but also for wheelchair users and children. You can wear them on the left or on the right leg. As a special feature, they are also available with a cotton backing, which further enhances the wearing comfort.

Capacity: Available in 0.6 litre or in 1.0 litre, it is intended for able-bodied people who do not usually have problems emptying the bag. Also, a smaller leg bag is not as conspicuous.

Inlet: The tube can be adapted to exactly suit your needs. This allows you to choose between wearing the bag on the thigh or on the calf. The enclosed loose adapter, smooth or stepped, is inserted into the (shortened) inlet tube of the bag. This way, you can connect it securely to all current condoms and catheters.

The cotton backing: Is made of natural cotton fleece. The leg bag is coated on one side only and remains transparent on the other side. As an alternative, you can use the SAUER cotton cover ► page 34.



Film with cotton backing



Textured film

100% testing

To increase security for the user, every bag is individually tested for leaks and functional capability.

Depending on their specification, SAUER urine collection bags show the following properties.

1 The connector

with or without universal adapter, this allows a simpler and safer connection for condoms with connecting tube, funnel condoms and catheters.

- no adapter: connect directly to SAUER condoms with connecting tubes.
- smooth adapter: connect to urinary condoms.
- stepped adapter: connect to catheters

2 The inlet tube

is hard or soft. As it comes in different lengths, it fits all body sizes.

3 The welded valve

allows for a smooth flow of urine, even with leg bags which do not lie flat.

4 The non-return valve

prevents urine backflow, for instance during physiotherapy sessions, and is important for infection control.

5 The eyelets

allow the straps to be fastened according to individual need, thus giving a more secure fitting.

6 The outlet tube

- swing tap: easy to handle, also with impeded dexterity of the hands, additional safety band available as protection against unintentional opening.
- turn tap: compact and discreet, particularly for small bags, collects to a night bag easily
- sliding tap: can be handled with only one hand; with this tap, the bag can be worn on the right or on the left leg.

7 The PVC film

is very strong and has a textured surface to prevent it from sticking to the skin.


8 Welding

The welded valve, the non-return valve and the outlet tap are welded securely to the film. The double welding makes it stronger when full.



Possibilities of use	W	A	C	
SAUER bendi bag.....without adapter.....1,3 l or 0,7 l	●	⊙	○	▶ page 22
SAUER bendi bag.....with fixed adapter.....1,3 l or 0,7 l	●	⊙	○	▶ page 23
SAUER bendi bag.....with push-on adapter.....1,3 l or 0,7 l	●	⊙	○	▶ page 24
SAUER leg bag.....with push-on adapter.....0,6 l	●	●	○	▶ page 25
SAUER leg bag.....with push-on adapter.....1,0 l	●	●	○	▶ page 26
Leg bag.....with fixed adapter.....0,6 l or 1,0 l	●	●	○	▶ page 27
SAUER sport bag.....without adapter.....0,5 l	⊙	●	○	▶ page 28
SAUER sport bag.....with fixed adapter.....0,5 l	⊙	●	○	▶ page 28
SAUER discreet bag.....with fixed adapter.....210 ml	⊙	●	○	▶ page 29
SAUER functional bag.....with fixed adapter.....2,0 l	⊙	⊙	●	▶ page 30
Night bag.....with fixed adapter.....1,5 l to 2,6 l	○	○	●	▶ page 31, 32

W = Wheelchair users A = able-bodied persons C = care in bed ● = suitable ⊙ = restricted usage ○ = not suitable

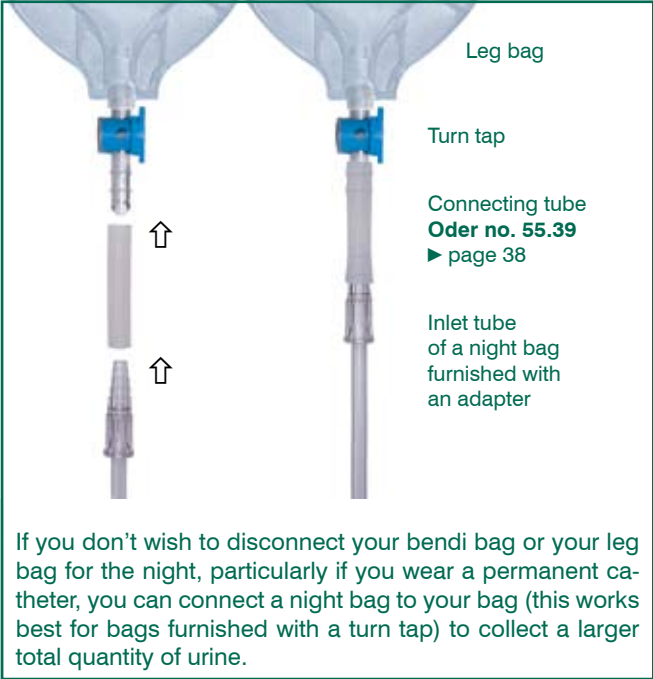

SAUER cotton cover	▶ page 34
Leg straps and waist belts	▶ page 35, 36
Leg bag holsters	▶ page 36
Accessories for night bags	▶ page 37
Accessories and spare components	▶ page 38, 39

Custom made SAUER urine collection bags


We have designed our range of products for bladder incontinence care to cover every need, yet they can also be considered as standard. We endeavour to constantly improve our products and find individual solutions wherever possible.

If you have any problems or questions regarding our standard range of products, please call us. We can advise you fully and try to find the best solution for your specific demand.

Inlet and outlet tube

Bendi bags in particular can be manufactured to meet your special requirements. However, we must sometimes charge a small fee for this. We can make the following alterations:

- shorten or lengthen the inlet tube (see picture on the right)
- fit an outlet tube instead of a swing tap, turn tap or sliding tap

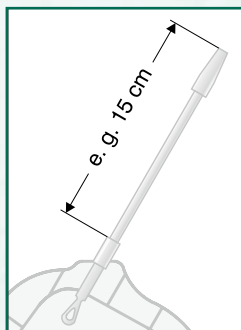

How to measure the length of the inlet tube

Swing tap for lefthand users

For these customers we offer the bendi bag (► pages 22–24) with a swing tap turned around by 180°. This way the tap lies on the inside of the leg and is protected from unintended opening. For easy distinction, these bags are equipped with yellow tap “wings”.

Cotton backing

To prevent the leg bag from coming into direct contact with the skin, we can, for a supplementary charge, cover the leg bag with a layer of cotton material. For technical reasons, this cover can only be applied on one side of the bag. This means that these bags are only suited for right-handers.

We advise you to choose the cotton cover instead, as it can be worn on either leg. Furthermore, it can be used frequently and is washable. This method is more economical and environmentally friendly.

Sterile collection bag

A variety of sterile collection bags are usually on stock (► page 33). On request, we can supply all bags in a sterile version and individually packed.

How to order

Custom-made products have a delivery time of approx. 14 days; sterile bags, if not available off the shelf, have a minimum delivery time of 8 weeks.

There will be an extra charge for all custom-made bags. We can neither exchange nor take back.

SAUER bendi bag without adapter

For wheelchair users. To be connected directly to our condoms with connecting tube (97.xx , 97.xxxx.xx, 56.xx, 53.xx). This means that no adapter is necessary. You can also cut the tube to fit your size.



- Leg hugging shape; welded angled inlet tube; capacity of 1.3l or 0.7l
- **Inlet:** without adapter; hard tube, 20 cm, can be shortened; non-return valve
- **Outlet:** swing tap, turn tap or sliding tap

	swing tap		turn tap		sliding tap	
hard inlet 20 cm adaptable length 						
no adapter	701.1020-xx	702.1020-xx	701.3020-xx	702.3020-xx	701.2020-xx	702.2020-xx







- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 701.1020-30).
- Fixation material for bags: leg straps ► page 35, cotton cover ► page 34.
- Tube cutter to adapt the length of the inlet tube ► page 38.
- Sterile urine collection bags ► page 33.






SAUER bendi bag with fixed adapter

For wheelchair users. To be connected to all urinary condoms and catheters.

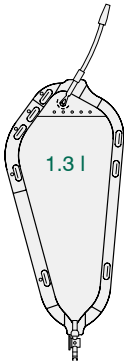- Leg hugging shape; welded angle inlet tube; capacity of 1.3l or 0.7l
- **Inlet:** fixed adapter, smooth or stepped; flexible tube, 12 cm or 22 cm, cannot be shortened; non-return valve
- **Outlet:** swing tap, turn tap or sliding tap

	swing tap		turn tap		sliding tap							
flexible inlet 12 cm length												
Notice: The smooth adapter has been approved of for usage with condoms, the stepped adapter is suitable for usage with catheters.												
smooth adapter	701.1112-xx	702.1112-xx	701.3112-xx	702.3112-xx	701.2112-xx	702.2112-xx						
stepped adapter	701.1212-xx	702.1212-xx	701.3212-xx	702.3212-xx	701.2212-xx	702.2212-xx						

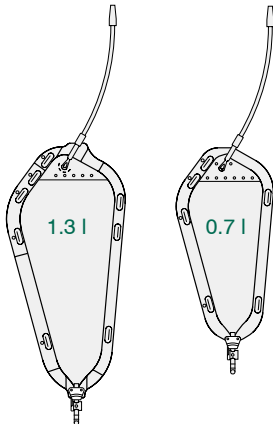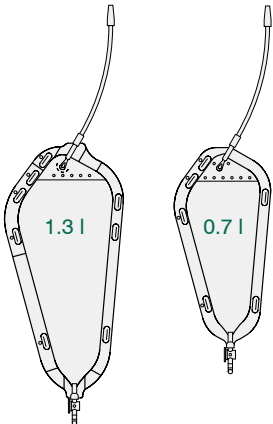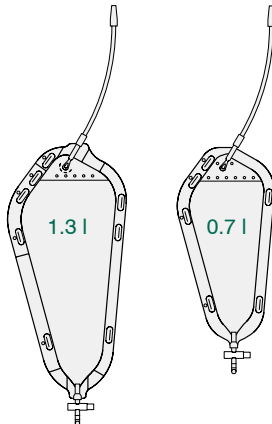	swing tap		turn tap		sliding tap	
flexible inlet 22 cm length						
Notice: The smooth adapter has been approved of for usage with condoms, the stepped adapter is suitable for usage with catheters.						
smooth adapter	701.1122-xx	702.1122-xx	701.3122-xx	702.3122-xx	701.2122-xx	702.2122-xx
stepped adapter	701.1222-xx	702.1222-xx	701.3222-xx	702.3222-xx	701.2222-xx	702.2222-xx

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 701.1112-30).
Fixation material for bags: leg straps ► page 35, cotton cover ► page 34, sterile urine collection bags ► page 33.

SAUER bendi bag with push-on adapter



For wheelchair users. To be connected to all urinary condoms and catheters.

- Leg hugging shape; welded angle inlet tube; capacity of 1.3l or 0.7l
- **Inlet:** push-on adapter, smooth or stepped; flexible tube, 35 cm, can be shortened; non-return valve
- **Outlet:** swing tap, turn tap or sliding tap

	swing tap		turn tap		sliding tap	
<div>flexible inlet 35 cm adaptable length</div> <div></div> <div>Notice: The smooth adapter has been approved of for usage with condoms, the stepped adapter is suitable for usage with catheters.</div>						
smooth adapter	701.1335-xx	702.1335-xx	701.3335-xx	702.3335-xx	701.2335-xx	702.2335-xx
stepped adapter	701.1435-xx	702.1435-xx	701.3435-xx	702.3435-xx	701.2435-xx	702.2435-xx





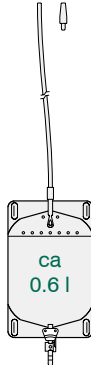
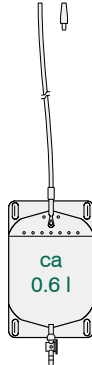
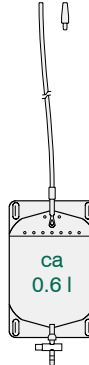
- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 701.1335-30).
- Fixation material for bags: leg straps ► page 35, cotton cover ► page 34.
- Tube cutter to adapt the length of the inlet tube ► page 38.
- Sterile urine collection bags ► page 33.

SAUER leg bag with push-on adapter

All-purpose bags for the able-bodied, wheelchair users and children. Can be connected to all urinary condoms and catheters. Can be worn either on the thigh (tube 15cm long) or on the lower leg (tube 45 cm long).


- Rectangular shape; straight inlet tube; capacity of 0.6l or 1.0l (► page 26), cotton backing on one side
- **Inlet:** push-on adapter, smooth or stepped; flexible tube, 15 or 45 cm, can be shortened;
- **Outlet:** swing tap, turn tap or sliding tap

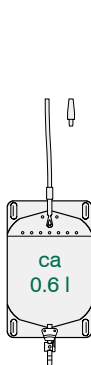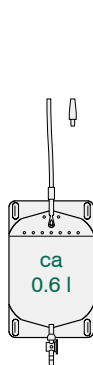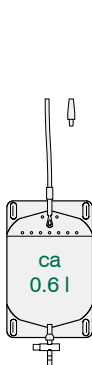

	swing tap		turn tap		sliding tap	
flexible inlet 15 or 45 cm adaptable length						
						
	Notice: The smooth adapter has been approved of for usage with condoms, the stepped adapter is suitable for usage with catheters.					
cotton backing						
smooth adapter	710.1315-xx	710.1345-xx	710.3315-xx	710.3345-xx	710.2315-xx	710.2345-xx
stepped adapter	710.1415-xx	710.1445-xx	710.3415-xx	710.3445-xx	710.2415-xx	710.2445-xx
plain film						
smooth adapter	730.1315-xx	730.1345-xx	730.3315-xx	730.3345-xx	730.2315-xx	730.2345-xx
stepped adapter	730.1415-xx	730.1445-xx	730.3415-xx	730.3445-xx	730.2415-xx	730.2445-xx


- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 710.1315-30).
- Fixation material for bags: leg straps ► page 35. For bags without cotton backing: cotton cover ► page 34, leg bag holsters ► page 36.
- Tube cutter to adapt the length of the inlet tube ► page 38.
- Sterile urine collection bags ► page 33.

flexible inlet
45cm
adaptable length




Notice: The smooth adapter has been approved of for usage with condoms, the stepped adapter is suitable for usage with catheters.


swing tap

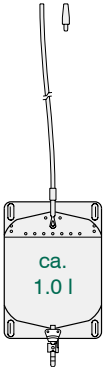

turn tap

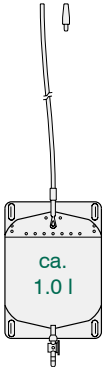

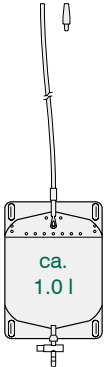

sliding tap



cotton backing			
smooth adapter	711.1345-xx	711.3345-xx	711.2345-xx
stepped adapter	711.1445-xx	711.3445-xx	711.2445-xx
plain film			
smooth adapter	731.1345-xx	731.3345-xx	731.2345-xx
stepped adapter	731.1445-xx	731.3445-xx	731.2445-xx

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 711.1345-30).
- Fixation material for bags: leg straps ► page 35. For bags without cotton backing: cotton cover ► page 34, leg bag holsters ► page 36.
- Tube cutter to adapt the length of the inlet tube ► page 38.
- Sterile urine collection bags ► page 33.





26

Leg bag with fixed adapter

All-purpose bags for the able-bodied, wheelchair users and children. Can be connected to all urinary condoms and catheters. Can be worn either on the thigh (tube 8 cm long) or on the lower leg (tube 32 cm long).

- Rectangular shape; straight or diagonal inlet tube; capacity of 0.6 l or 1.0 l
- **Inlet:** fixed adapter with a protection cap; 8 or 32 cm tube which cannot be shortened; non-return valve
- **Outlet:** Push-pull valve or sliding tap



Inlet	straight		diagonal	
approx. 8 or 32 cm cannot be shortened				
	stepped adapter	780.8811-xx	780.8835-xx	782.8811-xx
				783.2835-xx

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 780.8811-30).
- Fixation material for bags: leg straps ► page 35, cotton cover ► page 34, leg bag holsters for bags with 0.6 l ► page 36.
- Sterile urine collection bags ► page 33.

Note: The tube and the tap are not ring welded as with the SAUER leg bags. For this reason slight leakages might appear when stored for a longer period due to material fatigue. By individually testing the bags we can guarantee that the products are leakproof at the time of dispatching.

! The range of leg bags shown on this page represent the current situation, but it is very likely to undergo changes in near future. We'll keep on offering you bags of a comparable type, but they may vary regarding their specifications.

SAUER sport bag with or without fixed adapter

The sport bag responds to today's need for more flexibility in continence care. Because of their size, they can easily be worn on the thigh under shorts or swimming trunks.



For wheelchair users, able-bodied people and children. Without adapter to be connected to condoms with connecting tube (97.xx, 97.xxxx.xx, 56.xx, 53.xx); with adapter to be connected to all other urinary condoms and catheters.

- Straight or diagonal inlet tube; capacity of 0.5 l
- **Inlet:** with or without fixed adapter, smooth or stepped; non-return valve
- **Outlet:** swing tap or turn tap

	swing tap		turn tap	
Inlet	straight	diagonal	straight	diagonal
no adapter	720.1000-xx	721.1000-xx	720.3000-xx	721.3000-xx
smooth adapter	720.1100-xx	721.1100-xx	720.3100-xx	721.3100-xx
stepped adapter	720.1200-xx	721.1200-xx	720.3200-xx	721.3200-xx

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 720.1000-30).
- Fixation material for bags: leg straps ► page 35, cotton cover ► page 34.
- Sterile urine collection bags ► page 33.

The sport bags can be worn in the following ways:

- with a wide leg strap with Velcro® strap (57.44, 57.45 or 57.46) directly on the thigh, without cotton cover
- in a cotton cover (70.08) that is attached to the thigh strap with a Velcro® band



SAUER discreet bag with fixed adapter

The discreet bag is the smallest leg bag we can offer you. Its capacity of 210ml is sufficient for people with drip incontinence, but it can also be used during sport or swimming as inconspicuous interim solution. This small bag responds to today's need for more flexibility in incontinence care. Because of its small capacity it can easily be worn on the thigh, e.g. under shorts or swimming trunks, or directly in the underpant.



For wheelchair users, able-bodied people and for children. Connects to all urinary condoms and catheters.

- Diagonal inlet; capacity of 210ml
- **Inlet:** with fixed adapter, smooth or stepped; non-return valve
- **Outlet:** swing tap or turn tap

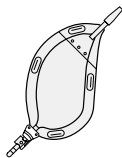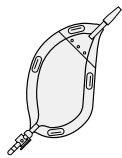	swing tap	turn tap
	 210 ml	 210 ml
smooth adapter	723.1100-xx	723.3100-xx
stepped adapter	723.1200-xx	723.3200-xx

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 723.1100-30).
- Fixation material for bags: leg straps ► page 35, cotton cover ► page 34.
- Sterile urine collection bags ► page 33.

You can carry the discreet bag in the following ways:

- with a wide leg strap furnished with a Velcro® loop (57.44, 57.45) directly on the thigh
- in the cotton cover (70.08) that is attached to the thigh strap (57.47) with a Velcro® strap
- without strap in the drawers







SAUER functional bag with fixed adapter

i We have designed the functional bag to fulfil a demand for night bags with high capacity that can also be attached to the leg when the wearer is up for a short time. Furthermore, the different types of outlet guarantee a higher degree of independence for people suffering from restricted dexterity of the hands. The sterile version of the SAUER Functional Bag is also suited for use with permanent catheters.



For use at night and for bedridden people, with higher capacity. Connects well to all urinary condoms and catheters.

- Rectangular shape; straight inlet; capacity 2,0l, with scale
- **Inlet:** fixed adapter, smooth or stepped; tube either 90 cm or 120 cm, cannot be shortened; non-return valve
- **Outlet:** Swing tap, turn tap or sliding tap

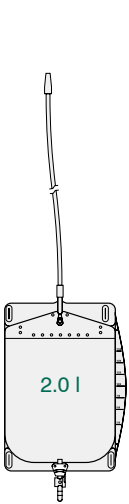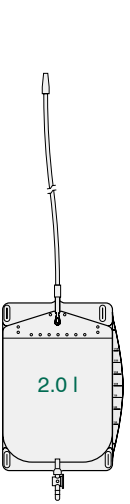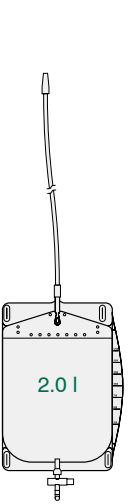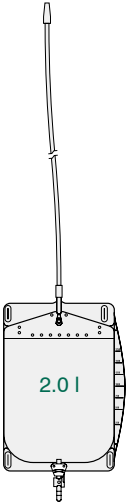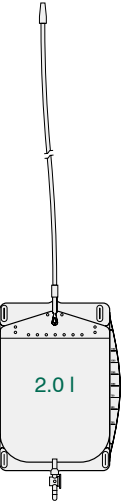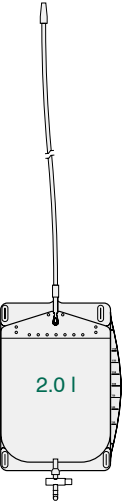	swing tap		turn tap		sliding tap	
Soft inlet 90 cm or 120 cm length						
						
Notice: The smooth adapter has been approved of for usage with condoms, the stepped adapter is suitable for usage with catheters.	2.0 l		2.0 l		2.0 l	
	2.0 l		2.0 l		2.0 l	
smooth adapter	771.11090-xx	771.11120-xx	771.31090-xx	771.31120-xx	771.21090-xx	771.21120-xx
stepped adapter	771.12090-xx	771.12120-xx	771.32090-xx	771.32120-xx	771.22090-xx	771.22120-xx


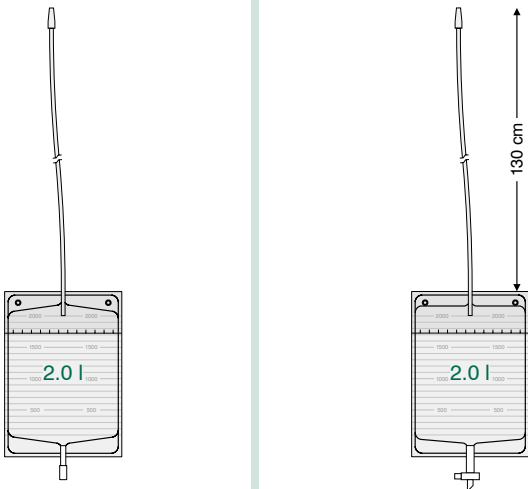
- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 771.11090-30).
- Fixation material for bags: leg straps ► page 35, bed fixture and night bag cover ► page 37.
- Sterile urine collection bags ► page 33.



Night bag with fixed adapter

For use during the night and for bedridden people. High capacity bag. To be connected to all urinary condoms and for intermittent catheterisation.

- Rectangular shape; straight inlet tube; capacity of 1.5l or 2.0l, with graduation
- **Inlet:** with fixed adapter, stepped; 90 cm or 130 cm tube, cannot be shortened; with non-return valve
- **Outlet:** with push-pull tap or sliding tap

with non-return valve		
	push-pull tap	sliding tap
<p>Inlet 90 cm or 130 cm length</p> <p>Note: These night bags are suitable for use with permanent catheters because of their non-return valve and outlet.</p>		
stepped adapter	790.8890-xx	791.8890-xx
		791.28130-xx

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 790.8890-30).
- Fixation material for bags: bed fixture and night bag cover ► page 37.
- Sterile urine collection bags ► page 33.







Note: Because of the low reimbursement prices night bags are not individually tested, so we cannot give a 100% guarantee they will be leakproof. We advise the use of a cover for extra safety ► page 34.

Night bag with fixed adapter – without outlet

For use during the night and for bedridden people. High capacity bag. To be connected to all urinary condoms and for intermittent catheterisation.



- Rectangular shape; straight inlet tube; capacity of 1.5l, 2.0l or 2.6l, with graduation (except 792.0890.1-xx)
- **Inlet:** with fixed adapter, stepped; 90 cm or 120 cm tube, cannot be shortened; with and without non-return valve
- **Outlet:** no outlet

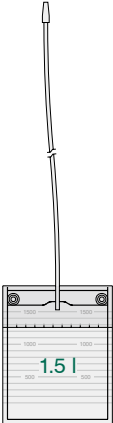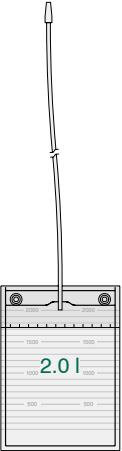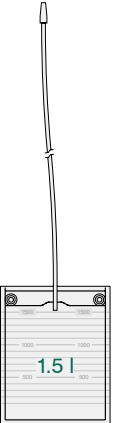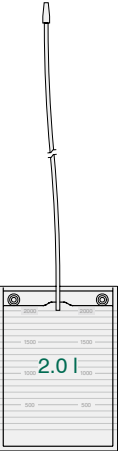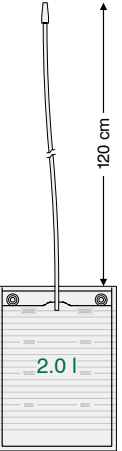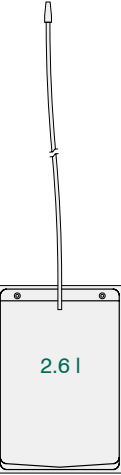	with non-return valve		without non-return valve			
	no outlet		no outlet			
Inlet 90 cm or 120 cm length						
	stepped adapter	790.0890-xx 791.0890-xx	790.0890.1-xx 791.0890.1-xx	791.08120.1-xx	792.0890.1-xx	

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 790.0890-30).
- Fixation material for bags: bed fixture and night bag cover ► page 37.
- Sterile urine collection bags ► page 33.

Note: Because of the low reimbursement prices night bags are not individually tested, so we cannot give a 100% guarantee they will be leakproof. We advise the use of a cover for extra safety ► page 34.

Sterile urine collection bags

The listed bags are available off the shelf, all other bags can be delivered as a sterile version (extra charge ► see page 20).

Sort of bag	C	Outlet	Adapter	Tube	Order no.	P
SAUER bendi bag	1.3 l	swing tap	smooth adapter	12 cm	701.1112-xx.S	23
	1.3 l	swing tap	stepped adapter	12 cm	701.1212-xx.S	23
	1.3 l	sliding tap	smooth adapter	12 cm	701.2112-xx.S	23
	1.3 l	sliding tap	stepped adapter	12 cm	701.2212-xx.S	23
	1.3 l	turn tap	stepped adapter	12 cm	701.3212-xx.S	23
	1.3 l	swing tap	stepped adapter	22 cm	701.1222-xx.S	23
	1.3 l	sliding tap	smooth adapter	22 cm	701.2122-xx.S	23
	1.3 l	sliding tap	stepped adapter	22 cm	701.2222-xx.S	23
	1.3 l	turn tap	stepped adapter	22 cm	701.3222-xx.S	23
	1.3 l	sliding tap	smooth adapter	35 cm, adaptable length	701.2335-xx.S	24
	1.3 l	sliding tap	stepped adapter	35 cm, adaptable length	701.2435-xx.S	24
	1.3 l	turn tap	stepped adapter	35 cm, adaptable length	701.3435-xx.S	24
	0.7 l	sliding tap	stepped adapter	12 cm	702.2212-xx.S	23
	0.7 l	turn tap	stepped adapter	35 cm, adaptable length	702.3435-xx.S	24
SAUER leg bag with cotton backing	0.6 l	swing tap	stepped adapter	15 cm, adaptable length	710.1415-xx.S	25
	0.6 l	swing tap	stepped adapter	45 cm, adaptable length	710.1445-xx.S	25
	0.6 l	turn tap	stepped adapter	45 cm, adaptable length	710.3445-xx.S	25
	1.0 l	turn tap	stepped adapter	45 cm, adaptable length	711.3445-xx.S	26
SAUER sport bag	0.5 l	turn tap	stepped adapter straight inlet	–	720.3200-xx.S	28
	0.5 l	turn tap	stepped adapter diagonal inlet	–	721.3200-xx.S	28
SAUER discreet bag	210 ml	turn tap	stepped adapter diagonal inlet	–	723.3200-xx.S	29
SAUER functional bag	2.0 l	turn tap	stepped adapter	90 cm	771.32090-xx.S	30
	2.0 l	turn tap	stepped adapter	120 cm	771.32120-xx.S	30
Night bag	2.0 l	push-pull tap	stepped adapter	90 cm	791.8890-xx.S	31

C = Capacity, P = Page in catalogue for further information

- Available in boxes of 10, 30 or 50. Please replace the “xx” within the order no. for the size of the unit (e.g. 701.1112-30).
- Fixation material for bags: leg straps ► page 35, cotton cover ► page 34, leg bags holsters ► page 36, bed fixture and cover ► page 37, tube cutter ► page 38.

SAUER cotton cover

i To avoid direct contact of the urine collection bag with the skin, the bag can be carried in a skin-friendly cotton cover (made of natural cotton, free of bleaching and dyeing substances). The SAUER cotton cover can be used frequently and is washable – an inexpensive and environmentally friendly solution.

The cotton cover doesn't disturb the fixation of the bag to the leg with the corresponding straps. You can wear the cotton cover on the left or on the right leg.

1 piece	for SAUER bendi bag 1,3 l	► page 22–24	Order no. 70.13
	for SAUER bendi bag 0,7 l	► page 22–24	70.12
	for SAUER leg bag 0,6 l	► page 25, 27	70.09
	for SAUER leg bag 1,0 l	► page 26, 27	70.10
	for SAUER sport bag 0,5 l	► page 28	70.08
	for SAUER discreet bag 210 ml	► page 29	70.08



Leg straps and waist belts

Wide leg strap – Original

60 cm long, 4 cm wide, elastic strap with Velcro® fastener. A second Velcro band (2 cm wide) helps to secure the leg bag onto the strap. For usage with bendi bags or leg bags.

1 piece

Order no. 57.46

Wide leg strap – Soft

Same as above, but with extra soft cotton padding. (Not illustrated)

1 piece

Order no. 57.45

Wide leg strap – Elasticated

Same as 57.46, but with non-slip silicone lines on the reverse side. The whole strap is made from a soft woven elasticated fabric. With loop for easy opening.

1 piece

Order no. 57.44

Wide leg strap – Sport bag

60 cm long, 4 cm wide, elastic strap with Velcro® fastener. Refer to ► page 34 for how to secure the SAUER cotton cover for sport bags.

1 piece

Order no. 57.47

Narrow leg strap

55 cm long, 2 cm wide, elastic strap with Velcro® fastener. Snap fasteners help secure the leg bag. For usage with bendi bags or leg bags.

2 snap fasteners.

1 piece

Order no. 57.48

Narrow leg strap – Elasticated

Same as 57.48, but with non-slip silicone lines on the reverse side. The whole strap is made from a soft woven elasticated fabric. Snap fasteners have to be ordered separately.

1 piece

Order no. 57.49

Latex leg strap

50 cm long, 2 cm wide, with 2 snap fasteners.

1 piece


Order no. 57.51

Snap fastener

To close the latex leg strap (57.51) or to secure the narrow leg strap (57.48) onto the leg bag.

1 piece

Order no. 57.53

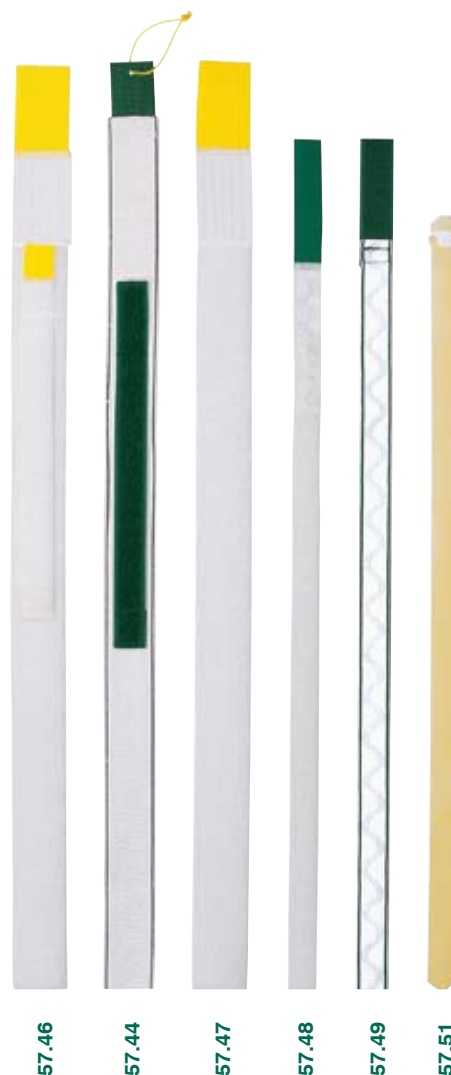

57.53

Waist belt

Made of cotton, serves as additional “anchor” for leg bags with leg-hugging shape and for the fixation of leg bag holsters.

1 piece	100 cm	Order no. 68.10
1 piece	150 cm	Order no. 68.15



68.10
68.15

Leg bag holsters

Particularly for people who are able to walk a holster can be used to carry a leg bag with a capacity of up to 0.6l. Holsters are available in several sizes and are made of a skin-friendly, elastic, fabric. They are washable. All holsters are furnished with a hole for the outlet tap in the lower rim, so that you can easily empty the bag.

Leg bag carrier – Netti

Shaped like “half a pair of underpants”. With belt and stitched-on bag for leg bags. Can be worn either on the left or on the right leg (different types). For leg bag 0.6l ► pages 25 and 27.

		for left leg	for right leg
1 piece	XS	Order no. ZNT-4	Order no. ZNT-4R
	S	ZNT-1	ZNT-1R
	M	ZNT-2	ZNT-2R
	L	ZNT-3	ZNT-3R
	XL	ZNT-5	ZNT-5R

Netti		
Size	Waist width	Knee width
Extra Small	60–70 cm	26–34 cm
Small	70–80 cm	28–38 cm
Medium	80–90 cm	30–44 cm
Large	90–100 cm	32–50 cm
Extra Large	> 100 cm	34–56 cm
Carebag Urobag		
Size	Thigh width	Thigh width
Small (S)	24–39 cm	30–45 cm
Medium (M)	36–55 cm	36–60 cm
Large (L)	40–70 cm	45–75 cm
Extra Large (XL)	–	54–90 cm

Leg bag carrier – Carebag


Shaped like a cuff. With attached bag for leg bags. Kept in position by elastic bands on the upper and lower rim. Can be worn on the left or right leg. For leg bags 0.6l ► page 25 and 27.

1 piece	Small	Order no. Z911.58
	Medium	Z912.58
	Large	Z913.58

Leg bag carrier – Urobag **NEW**

Tailored like a gaiter, with Velcro® fastener and attached pocket. The elastic collar holds the bag in place. Can be worn on the left and the right thigh or lower leg. For leg bags 0.6l ► page 25 and 27.

2 pieces	Small	Order no. Z91-01-230
	Medium	Z91-02-430
	Large	Z91-03-630
	X-Large	Z91-04-730

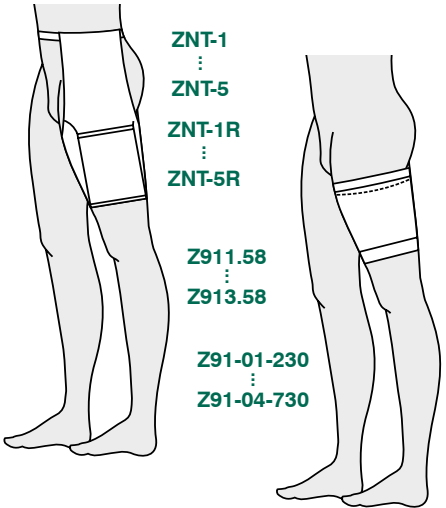

Accessories for night bags

Plastic bed fixture

With round hanger. Durable material, blue. The hangers can be folded to the side for transportation.

Can hold a bag with a capacity up to 2 litres (2 kg).

1 piece

Order no. 70.51




70.51

Wire bed fixture


With round hanger. Made of wire varnished steel. Can hold a bag with a capacity up to 3 litres (3 kg).

1 piece

Order no. 70.50

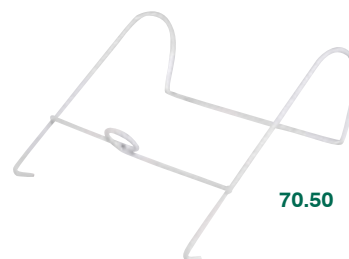

70.50

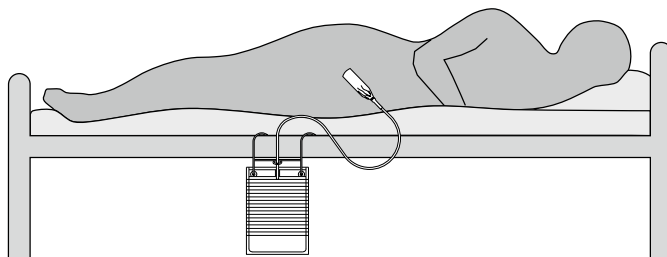

Night bag cover

The night bag is simply inserted into the waterproof cover for added safety and sight protection. Suitable for all night bags with a capacity of up to 2 litres. With night bags furnished with an outlet you can fold the tap up.

1 piece

Order no. 70.52



70.52

Accessories and spare components

Stopper

To close condoms, catheters, and tubes.

	5 pieces	Order no. 60.07
sterile	5 pieces	Order no. 60.07.S



Tubing

Soft PVC, Ø 7.3×5.3 mm	☒	Order no. 55.00
Latex, Ø 9.0×6.0 mm, 5 cm long (length as required)	☒	Order no. 55.06




Tube cutter *NEW*

To adapt the length of the inlet tube.	1 piece	Order no. 55.11
Spare blades (not illustrated)	5 pieces	☒ Order no. 55.12

Connecting tube

Silkolatrix® Ø 9×6.0 mm, 5 cm long	5 pieces	Order no. 55.36
Latex Ø 9.0×6.0 mm, 5 cm long	5 pieces	Order no. 55.38
Silicone Ø 6.0×9.0 mm, 5 cm long	5 pieces	Order no. 55.39

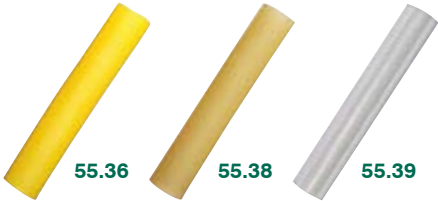

Tube coupling

Ø 6.5×11.5 mm	1 piece	Order no. 62.02
Ø 8.5×11.5 mm	1 piece	Order no. 62.04
Ø 6.0 mm	1 piece	Order no. 62.06



Y piece

Enables simultaneous usage of 2 urine collection bags, sterile.	
	1 piece Order no. 62.10.S

Adapter

To connect the condom or catheter to a urine bag with a soft, adjustable inlet tube.

Push-on smooth adapter	5 pieces	Order no. 60.09
Push-on stepped adapter	5 pieces	Order no. 60.10
Push-on stepped adapter, sterile	5 pieces	Order no. 60.10.S



☒ The article cannot be returned.

Swing tap	5 pieces	Order no. 57.67
Turn tap	5 pieces	Order no. 57.68
Sliding tap	5 pieces	Order no. 57.66

Safety ring

For swing tap to prevent accidental opening.

5 pieces **Order no. 55.31**

Catheter valve

To close a catheter.

- no tube coupling

Swing tap with stepped adapter	5 pieces	Order no. 60.20
Sliding tap with stepped adapter	5 pieces	Order no. 60.26
Sliding tap with stepped adapter, sterile	5 pieces	Order no. 60.26.S
Turn tap with stepped adapter	5 pieces	Order no. 60.31
Turn tap with stepped adapter, sterile	5 pieces	Order no. 60.31.S

- with tube coupling

Swing tap with stepped adapter, sterile	5 pieces	Order no. 60.21.S
Turn tap with stepped adapter	5 pieces	Order no. 60.33
Turn tap with stepped adapter, sterile	5 pieces	Order no. 60.33.S


With connecting tubes, the part that is used for connection to tubes, condom outlets and catheters is stepped. With adapters and catheter valves it is either stepped or smooth.



Connecting tube
stepped



Smooth adapter

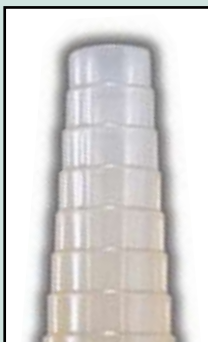

Stepped adapter

■ Further spare parts on request.



57.67



57.68



55.31



57.66



60.20



60.26
60.26.S



60.31
60.31.S



60.21.S



60.33
60.33.S

SAUER drip urinal

i For people suffering from partial incontinence (for instance after prostate removal). Designed to hold small amounts of urine for short periods. Also particularly suitable for swimming since the adhesive is waterproof. This urinal is furnished either with self-adhesive urinary condoms “SAUER-Comfort” or with latex condoms that are attached with skin adhesive or double-sided adhesive strips. The self-adhesive condoms are particularly suitable when aiming at a quick and safe solution with as little effort and accessories as possible. The 2-piece condoms allow for a more individual adaptation, as the strength and amount of glue can be chosen regarding the skin conditions. Additionally the adhesive and buffer zone can be positioned to meet individual requirements. The urinal can be closed and emptied with a turn tap or a sliding tap.

The complete months supply for partial incontinence. Condoms, adhesive or strips and taps can be chosen from our range to suit your individual demands.

SAUER drip urinal with self-adhesive condoms – SAUER-Comfort

- 30 condoms
- 1 protective cloths to go with the size of the condom
- 30 turn taps or 30 sliding taps

				Self-adhesive condom					
				97.30					
1	with turn tap	23	+	97	+	30	=	23.97.30	
2	with sliding tap	21	+	97	+	30	=	21.97.30	

The order-no. is composed according to the diagram. The last two figures represent the condom diameter (in mm).

SAUER drip urinal with latex condoms – Original/Extra thin

- 30 condoms
- 1 protective cloths to go with the size of the condom
- 2 tubes of skin adhesive or 30 doubele-sided adhesive tapes
- 30 turn taps or 30 sliding taps

				Skin adhesive		Condom			
				50.01		53.30			
3	with turn tap	23	+	01	+	30	=	23.01.30	
4	with sliding tap	21	+	01	+	30	=	21.01.30	

The order no. is composed according to the diagram. The last two figures represent the condom diameter (in mm). For the extra-thin condoms you have to add a “D” to the end of the order no. (e.g. 23.01.230.D).



■ Turn taps and sliding taps can be individually ordered ► page 39.

Pocket size urinal Uribag



What do you do when you want to spend a penny and no toilet is available? Not only wheelchair users are faced with this situation but also crane drivers, glider pilots and motorists have the same problem.

The Uribag comes to the rescue in an emergency: secure and discreet – it can be folded away and re-used.

The Uribag is available in two different versions for men and women. Both have the following features in common:

- urine bag made of soft latex, can be folded away
- upper rim and cap are made of synthetic material
- closes securely
- high capacity of approx. 1 litres
- can be re-used

The difference between the two lies in the shape of the top part. For men this is cylindrical, for women it is adapted to their body shape.

Uribag for men 1 piece

☒ **Order no. 999.806**

Uribag F for women 1 piece

☒ **Order no. 999.808**

URibag



999.806



999.808

1 The cap closes the Uribag securely. With the Uribag for men it is secured by a connecting piece so that it can't get lost.

2 The tube is made of synthetic material.

♂ For men it has got a cylindrical shape (length approx. 8 cm, diameter 4 cm);

♀ For women the anatomical design fits comfortably.

3 The collector bag is made of soft latex and can hold 1,2 l of fluid.

☞ The collection bag can be folded away completely into the tube.

How to use it

Open the cap and take the bag out of the tube. After use wash in warm soapy water. Let the bag dry before folding.

☒ The article cannot be returned.

Latex urinal

Until 1980, the latex urinal was the standard system used for bladder incontinence. However, as a result of clinical tests, the latex urinal was replaced by a PVC (single-use) disposable system. The reason was not only the easier handling, but it reduces the additional sources of infection in clinics, prevents bladder infection, supports and aids the medical treatment of infected urine, and dispenses with the need for the necessary cleaning of the latex urinal.

For the condoms without connecting tubes we offer the same latex qualities and condom sizes as for the condoms with a connecting tube linked to the condom by a nipple (► page 11). They are individually tested and rolled up.

Latex condoms without connecting tube – Original

30 pieces	Size Ø 18 mm	Order no. 52.18
	⋮	⋮
	40 mm	52.40

Latex condoms without connecting tube – Extra thin

30 pieces	Size Ø 26 mm	Order no. 52.26.D
	⋮	⋮
	35 mm	52.35.D

Skin adhesive and double-sided adhesive tapes.....► page 12–14
Measuring tape, protective cloths, scissors► page 15
Adhesive strengthener and skin care► page 16

Leg bags made of Silkolatex®

For wheelchair users. Capacity between 0.3l and 0.8l. Inlet with non-return valve. Outlet with wing, swing or sliding tap. Latex straps with snap fasteners (57.50) help secure the leg bag.

1 leg bag 0.8l, 1 wing tap, 2 leg straps	Order no. 57.11	
1 leg bag 0.8l, 3 sliding taps, 2 leg straps	57.13	
1 leg bag 0.8l, 3 swing taps, 2 leg straps	57.14	
1 leg bag 0.6l, 1 wing tap, 2 leg straps	57.15	
1 leg bag 0.3l, 1 wing tap, 1 leg strap	57.16	
■ If needed, you can order the following parts separately ► page 43:		
Nipple	60.03	57.11
Inlet tube	55.08 (please order required length)	⋮
Upper connecting peace	61.01	57.16

Note: We advise you to alternately use two latex leg bags, so that there is enough time to disinfect while wearing the other ► page 46 for disinfectant products.



Accessories for the latex urinal

Waist belt made of cotton 100cm long	1 piece	Order no. 28.10
150cm long	1 piece	Order no. 28.15
Leg strap made of Silkolatex®, with snap fastener	1 piece	Order no. 57.50
Latex inlet tube Ø 8×12mm	<input checked="" type="checkbox"/>	Order no. 55.08
Ø 10×14mm	<input checked="" type="checkbox"/>	Order no. 55.10
(please order required length)		
Tube cutter to adapt the length of the inlet tube	1 piece	Order no. 55.11
Spare blades	5 pieces	Order no. 55.12
Connecting tube Ø 8.5×11.5mm	1 piece	Order no. 62.04
Ø 8.0mm	1 piece	Order no. 62.08
Plastic snap fastener for 57.50	1 piece	Order no. 57.52
Strap securing disc for latex leg bag	1 piece	Order no. 57.54
Wing tap for leg bag 57.11	1 piece	Order no. 57.56
for leg bag 57.15 and 57.16	1 piece	Order no. 57.57
Inlet valve for leg bag 57.11–57.16	1 piece	Order no. 57.60
Swing tap	5 pieces	Order no. 57.67
Turn tap	5 pieces	Order no. 57.68
Sliding tap	5 pieces	Order no. 57.66
Safety ring for swing tap	5 pieces	Order no. 55.31
Piercing stick	1 piece	Order no. 60.01
Plastic nipple green, for 55.08	1 piece	Order no. 60.03
red, for 55.10	1 piece	Order no. 60.05
Stopper for latex tube	5 pieces	Order no. 60.07
Upper connecting piece with rubber washer	1 piece	Order no. 61.01
Connecting piece for night bag	1 piece	Order no. 61.06
Cap for upper connecting piece	1 piece	Order no. 61.08

☒ The article cannot be returned.

We have developed the KIC-System for patients that are either incontinent between catheterisation intervals or that achieve continence only with medication, which they cannot or will not tolerate because of side effects. These people would have to put on a new condom several times a day after each catheterisation.

KIC-System

The solution for combining condoms and intermittent catheterisation.



In the KIC-System, we have a removable connector instead of a fixed tube to connect the leg bag. This connector can be removed from the urinary condom and, using the special condom expander tool, the condom can then be pulled over the penis shaft so that the penis tip is left free for disinfecting and catheterisation. Afterwards, the urinary condom can be restored to its original position and connected to the existing drainage system. This procedure can be repeated any number of times.

With the KIC-System, you only need one urinary condom per day – this not only protects the skin but it can also save a lot of money.

The KIC condom expander is made of sturdy plastic material and can be used over a longer period of time (for several months).

The KIC condoms have a reinforced outlet, which can be stretched considerably but always goes back to its original shape. The opening does not tear when handled with care. There is a choice of 11 sizes in the “Original” version and of 3 sizes in the “Extra thin” version.

The KIC connector replaces the connecting tube that the urinary condom is usually furnished with. This allows you to separate the inlet tube of the leg bag from the urinary condom in order to expand the condom with the condom expander tool and then pull it back over the penis tip.

Cleaning and disinfecting KIC condom expanders and KIC connecting tubes are easy to clean with common disinfectants. Just follow the disinfectant manufacturer’s instructions.

Aids for ISC

You can find further articles for bladder catheterisation in a separate catalogue or on our homepage:
www.sauercontinence.com



Advice

For further information about our KIC-System, please contact your local continence adviser for further information ► page 48.

KIC-System



The system consists of three components: the KIC condom expander, KIC condoms and the KIC connector. We provide sufficient connectors and connecting tubes with our KIC condoms. However, you can order them separately if necessary.

KIC condom expander

Made of sturdy plastic material that is easy to clean.

1 piece

Order no. 100.01

KIC condoms – Original

Furnished with a punched hole to position the KIC connector. Apart from this, these condoms have the same features as our standard latex condoms “Original”. Available in 11 sizes.

Available as a set with 30 condoms, 10 connectors and connecting tubes.

Size Ø 18 mm

Order no. 103.18

20 mm

103.20

22 mm

103.22

24 mm

103.24

26 mm

103.26

28 mm

103.28

30 mm

103.30

32 mm

103.32

35 mm

103.35

37 mm

103.37

40 mm

103.40

KIC condoms – Extra thin

Furnished with a punched hole to attach the KIC connector, the tip of the condom is reinforced. Apart from this, these condoms have the same features as our latex condoms “Extra thin”. Available in 3 sizes.

Available as a set with 30 condoms, 10 connectors and connecting tubes.

Size Ø 26 mm

Order no. 103.26.D

30 mm

103.30.D

35 mm

103.35.D

KIC connector

Made of plastic. Comes with a connecting tube made of latex.

10 pieces

Order no. 100.05

Connecting tube

To go with KIC connector 100.05. Made of latex.

5 pieces

Order no. 55.38



100.01



103.18

⋮

103.40

103.26.D

⋮

103.35.D



100.05

55.38

KIC

patented

Note: Skin adhesive or double-sided adhesive tapes must be ordered separately.

Barrycidal disinfectant

A quick and efficient disinfectant against bacteria, viruses and fungi. Neutralises odorous bacteria. Odourless. Free of mercury, formaldehyde, phenol or alcohol.

Barrycidal

Ready for use 1 percent solution.

1 litre

Order no. 90.13

Spray head

for the plastic bottle 90.13.

1 piece

Order no. 90.14

Barrycidal spray

With ready for use 1 percent solution.

50ml

Order no. 50.54

Sanitary wipes

In practical wipe dispenser.

100 wipes

Order no. 50.55



Barrydin disinfectant – concentrated solution

Suitable for disinfecting any urine drainage system, particularly the latex system. Effective against bacteria, fungi, viruses. Takes effect quickly and is material friendly. Comes in a plastic bottle.

Add water to get a ready-for-use solution (1.5%). Includes a measuring jug.

1 litre

Order no. 90.15



Please make sure to follow the instructions on the label carefully. Avoid skin contact!

Plastic funnel

For pouring the disinfectant into the urine drainage system.

1 piece

Order no. 90.20



Literature

i Sauer Continence offers you a complete range of medical aids for your individual bladder incontinence management. Our products are suited for persons in wheelchairs, able-bodied persons, persons confined to bed and persons with partial incontinence. Furthermore, we can offer you various sources of information (available in several languages, free of charge). Please turn to your local continence adviser for further information ► page 48.

Catalogue “Continence Systems”

Condoms, urine bags and accessories made by Sauer Continence.

Order no. 99.900

Catalogue “IQ-Cath Catheter & Accessories”

Single use catheter made by Sauer Continence.

Order no. 99.936

Tutorial “Continence Systems”

This tutorial contains further information on bladder paralysis and lots of advice on incontinence care and how to use our various urine drainage systems.

Order no. 99.890

DVD “Urine drainage system”

Demonstrates how to apply a urine drainage system: a carer helps to put a condom on; a tetraplegic puts the condom on himself; securing the leg bag; connecting the leg bag to the condom.

Order no. 99.932



99.900



99.936



99.890

With lots of practical hints!



99.932

You can also visit us on the Internet:
www.sauercontinence.com





SAUER CONTINENCE **World-wide**

Our products are distributed in many countries around the world. Please check our website www.sauercontinence.com and follow the link "International" for further information about your nearest dealer. In all other countries you can contact our export department for a direct delivery to your home.



Claudia Hofer



Uschi Birk-Meyer

Claudia Hofer

Tel. 06226 960 104

Fax 06226 960 010

claudia.hofer@manfred-sauer.de

Uschi Birk-Meyer

Tel. 06226 960 016

Fax 06226 960 010

uschi.birk-meyer@manfred-sauer.de



Please visit our website for further information: www.manfred-sauer.com



Index


100 % testing	4, 18	DVD "The Urine Drainage System"	47	Composition	5
Accessories and spare components	38, 39	Fixation material for bags	34–37	Skin care	16
Accessories for condoms	15	Functional bag	30	Skin-Prep	16
Accessories for night bags	37	Funnel	46	Sliding tap	39, 43
Accessories for the latex urinal	43	Inlet valve	43	Special skin adhesive	13
Adapter	38, 39	KIC condom expander	45	Sport bag	28
Adhesion effect	5	KIC condoms	45	Spray head	16, 46
Adhesive remover pads	13	KIC connector	45	Sterile urine collection bags	21, 33
Adhesive spray	13	KIC-System	44, 45	Stopper	38, 43
Adhesive enhancer	16	Latex articles	5	Strap securing disc	43
Adhesive zone	5–9	Latex condoms – Extra thin	11	Swing tap	39, 43
Aids for ISC	44	Latex condoms – Original	11	Synthetic condoms – Anti-Allergic	10
Anti-Allergic condoms	10, 40	Latex inlet tube	43	Tube	38, 43
Barrycidal – disinfectant	46	Latex urinal	42, 43	Tube coupling	38, 43
Barrydin – disinfectant	46	Leftdressers	21	Tube cutter	38, 43
Bed fixture	37	Leg bag holsters	36	Turn tap	39, 43
Bendi bag with fixed adapter	23	Leg bag made of Silkotex®	42	Tutorial "Continence Systems"	47
Bendi bag with push-on adapter	24	Leg bag with fixed adapter	27	U-Bond – skin adhesive	13
Bendi bag without adapter	22	Leg bag with push-on adapter	25, 26	UltraFlex – self-adhesive condoms	8
Buffer zone	4–8	Leg straps	35, 43	Upper connecting piece	43
Cap	43	Literature	47	Uribag – pocket size urinal	41
Carebag – leg bag carrier	36	Measuring tape	15	Urine collection bags	18–33, 42
Catalogue "Continence Systems"	47	Netti – leg bag carrier	36	Configuration	18, 19
Catalogue "IQ-Cath"	47	Night bag	31, 32	Custom products	21
Catheter valve	39	Night bag cover	37	Period of wear	20
Comfort – self-adhesive condoms	6, 7	Nipple	43	Possibilities of use	20
Condom adhesion	12–15	Non-return valve	19, 22–32	Urinary condoms	4–11, 40, 42, 45
Condoms → Urinary condoms		Period of wear for urine collection bags	20	2-piece condoms	10, 11, 40, 42, 45
Connecting piece	43	Piercing stick	43	Adhesion	12–14
Connecting tube		Plastic snap fastener	43	Choice	5
as accessory	38, 45	Pocket size urinal Uribag	41	Configuration	4
as part of condoms	4, 6–11, 42, 44	Pop-On – self-adhesive condoms	9	Custom products	4
Continence adviser	38	Preventox	16	Self-adhesive	4, 6–9
Cotton backing	18, 21, 25, 26	Protective cloths	15, 40	Urobag – leg bag carrier	36
Cotton cover	21, 34	Roller, Preventox	16	Uroclean – washing gel	16
Cranberry concentrate	17	Safety ring	39, 43	Waist belt	36, 43
Custom products	5, 21	Scissors	15	Washing gel Uroclean	16
Discreet bag	29	Self-adhesive condoms	4, 6–9	Wide-Band – self-adhesive condoms	9
Disinfectant	46	no buffer zone	8, 9	Wing tap	43
Double-sided adhesive tapes	14	Skin adhesive	5, 12, 13	Y piece	38
Drip urinal	40	Choice	12		

In 2001, the Manfred Sauer GmbH adopted a Quality Management System for the development, the manufacturing and the distribution of continence systems. The application of the standards DIN EN ISO 13485:2003 and DIN EN ISO 9001:2000 was verified and certified.




We reserve the right to make technical changes to our products and packaging which e.g. help to preserve the environment.

[illegible]

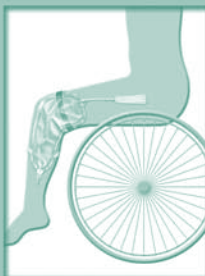

presented by

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

**SAUER
CONTINENCE**

MANFRED SAUER GMBH • Continenze-Systems • Neurott 7 • 74931 Lobbach • Germany
Tel. +49 (0)6226 960 104 • Fax +49 (0)6226 960 010 • export@sauercontinence.de • www.sauercontinence.com